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**1 - Description and intended use**

A partial or total shoulder prosthesis is comprised of various components, all of which are packaged individually and are designed to replace all or part of the natural shoulder joint. A total or partial shoulder arthroplasty is intended to reduce pain and improve the mobility of the patient's joint. The performance capabilities of the device and its service life are closely linked to a multitude of factors, which are dependent upon the patient (physiology, occupation) and upon the surgery undertaken by the surgical team (selection, fitting and fixing of components).

**2 - Materials**

| Components  | Fixation method        | Materials abbreviation                       | Reference standard                    |
|---|------------------------|--|---------------------------------------|
| Short humeral stem ISA stemless                           | Cementless             | Ti6 Al4 V-ELI + Ti                           | ISO 5832-3                            |
| Cementless Humeral Stem ISA onlay                         | Cementless             | Ti6 Al4 V-ELI + Ti                           | ISO 5832-3                            |
| Cemented Humeral Stem ISA onlay                           | Cemented               | Ti6 Al4 V-ELI or Ti6 Al4 V-ELI + TiN         | ISO 5832-3                            |
| Cementless Humeral Stem ISA inlay                         | Cementless             | Ti6 Al4 V-ELI + Ti                           | ISO 5832-3                            |
| Cemented Humeral Stem ISA inlay                           | Cemented               | Ti6 Al4 V-ELI or Ti6 Al4 V-ELI + TiN         | ISO 5832-3                            |
| Cementless Humeral Stem ISA fracture                      | Cementless             | Ti6 Al4 V-ELI + Ti + HAP                     | ISO 5832-3                            |
| Cementless Humeral Stem ISA revision                      | Cementless             | Ti6 Al4 V-ELI + Ti                           | ISO 5832-3                            |
| Dual taper  | Cementless or Cemented | Ti6 Al4 V-ELI                                | ISO 5832-3                            |
| Humeral baseplate ISA onlay                               | Cementless or Cemented | Ti6 Al4 V-ELI or Ti6 Al4 V-ELI + TiN         | ISO 5832-3                            |
| Humeral Augment ISA inlay                                 | Cementless or Cemented | Ti6 Al4 V-ELI or Ti6 Al4 V-ELI + TiN         | ISO 5832-3                            |
| Humeral head  | Cementless or Cemented | CoCr or Ti6 Al4 V-ELI or Ti6 Al4 V-ELI + TiN | ISO 5832-12 or ISO 5832-3             |
| Humeral liner   | Cementless or Cemented | UHMWPE and Ti6 Al4 V-ELI                     | ISO 5834-1 & ASTM F648 and ISO 5832-3 |
| Glénosphères  | Cementless or Cemented | CoCr or Ti6 Al4 V-ELI or Ti6 Al4 V-ELI + TiN | ISO 5832-12 or ISO 5832-3             |
| Pegged Glenoid  | Cemented               | UHMWPE and T40 + Ti                          | ISO 5834-1 & ASTM F648 and ISO 5832-2 |
| Cemented Keeled Glenoid                                   | Cemented               | UHMWPE and T40                               | ISO 5834-1 & ASTM F648 and ISO 5832-2 |
| Augmented Pegged Glenoid                                  | Cemented               | UHMWPE and T40 + Ti                          | ISO 5834-1 & ASTM F648 and ISO 5832-2 |
| Cementless Humeral baseplate                              | Cementless             | Ti6 Al4 V-ELI + Ti                           | ISO 5832-3                            |
| Cementless Glenoid Baseplate with Central Screw           | Cementless             | Ti6 Al4 V-ELI + Ti                           | ISO 5832-3                            |
| Cementless Augmented Glenoid Baseplate with Central Screw | Cementless             | Ti6 Al4 V-ELI + Ti                           | ISO 5832-3                            |
| Locked Peripheral Peg                                     | Cementless             | Ti6 Al4 V-ELI + Ti                           | ISO 5832-3                            |
| Cortical screws, Central screws, Obturator                | Cementless or Cemented | Ti6 Al4 V-ELI                                | ISO 5832-3                            |

**3 - Indications**

|                      | Intact glenoid humeral hemiarthroplasties  | Total shoulder arthroplasties performed with anatomic prostheses   | Total shoulder arthroplasties performed with inverted prostheses  |
|----------------------|--|--|---|
| →<br>Components<br>↓ | Humeral head, screws   | Glenoid, Humeral head, Dual taper  | Glenoid baseplate, Glenosphere, Humeral liner, Humeral baseplate, Humeral Augment, screws   |
| Stemless stems       | - Primary osteoarthritis<br>- Post-traumatic osteoarthritis<br>- Aseptic osteonecrosis   | - Gleno humeral osteoarthritis<br>- Primary gleno humeral osteoarthritis<br>- Post-traumatic osteoarthritis                                  |   |
| Onlay stems          |  | - Gleno humeral osteoarthritis<br>- Rheumatoid arthritis<br>- Osteoarthritis with glenoid posterior wear<br>- Osteoarthritis with B2 glenoid | - Primary Gleno humeral osteoarthritis<br>- Gleno humeral osteoarthritis grade 1 to 5<br>- Cuff tear arthropathy<br>- Massive rotator cuff tear<br>- Post-traumatic osteoarthritis<br>- Avascular necrosis<br>- Chronic instability<br>- Glenoid erosion E0 to E3<br>- Rheumatoid arthritis   |
| Inlay stems          |  | - Primary gleno humeral osteoarthritis   | - Gleno humeral osteoarthritis<br>- Primary rotator cuff deficiency<br>- Failure of rotator cuff operation<br>- Failure of arthroplasty<br>- Prosthetic instability<br>- Proximal humerus malunion  |
| Fracture stems       | - Humeral head displacement<br>- Three-part fractures<br>- Four-part fractures<br>- Proximal humeral fracture-dislocations<br>- Comminuted fractures of the humeral neck |  | - Acute proximal humeral fractures<br>- Fractures of the humeral neck<br>- Glenoid defects  |
| Revision stems       |  |  | - Gleno humeral osteoarthritis<br>- Cuff tear arthropathy<br>- Severe rotator cuff deficiency<br>- Failure of rotator cuff operation<br>- Failure of hemiarthroplasty<br>- Failure of total arthroplasty<br>- Humeral fracture<br>- Proximal humeral fracture and sequelae<br>- Shoulder pseudoparalysis<br>- Proximal humerus malunion |

Ti6 Al4 V-ELI and Titanium Nitride coated Ti6 Al4 V-ELI (Ti6 Al4 V-ELI + TiN) components are intended for allergic patients or for patients with suspected sensitivity to CoCr alloy constituents: nickel, manganese, chromium, cobalt and/or molybdenum.

**4 - Counter-indications**

The implantation of a total or partial shoulder prosthesis, in common with any surgical procedure, has attendant risks, specifically associated with anaesthesia and with the operation itself.

General risks:

- Systemic or local infections,
- Severe vascular or neurological deficiencies
- Bone loss or poor bone quality, which is liable to compromise the stability of the implant
- Allergies to materials present in the implant
- Bone tumour in the region of the implant

Anatomical prostheses:

- Severe deficiency of the cuff
- Insufficient bone stock for the attachment of implants
- Permanent instability

**5 - Warnings and precautions**

Prior to any clinical use, the surgeon must be fully versed in all aspects of the operation, the implants and associated instruments together with their applications and limits.

Surgeons must notify their patients of the limitations of the prosthesis, and of the associated impact upon their activities (professional, sporting or leisure activities), together with the service life of the prosthesis. Patients must also be informed of the need to avoid strenuous effort before attachment and scarring is complete.

Repeated and/or excessive stresses may result in the defective attachment of the implant.

Every precaution must be taken to avoid damage to prosthetic components during fitting. Particular attention must be paid to cement particles and/or bone fragments.

In all cases, the prosthesis will not provide the same function as a normal and healthy natural shoulder joint.

Any abnormal pain, unexpected restriction in the amplitude of movement, or variation in the performance of the prosthesis further to attachment and the completion of rehabilitation, must be referred to a medical practitioner.

Components of the ISA™ MOVE-UP shoulder prosthesis are not designed for use in conjunction with the components of other manufacturers.

MOVE-UP implants must be fitted using MOVE-UP instruments which are designed for this purpose.

MOVE-UP prosthetic components are single-use components, and must never be reconditioned, re-sterilized, modified or altered in any way. Any re-implantation of an implant will expose the patient to biological risks which may have severe consequences.

No test for the evaluation of the safety and compatibility of the implant system in an MRI environment (Magnetic Resonance Imaging) has been conducted for the evaluation of associated risks. These include the potential heat-up and/or migration of metallic components. Image artifacts, such as signal drop-out zones and distortions may occur, particularly in the area immediately surrounding the implant.

Any severe incident arising in conjunction with the prosthesis must be reported to MOVE-UP and to the competent authority of the country in which the surgeon and/or the patient is established.

**6 - Associations**

Key:

C = Compatible (*diametral mismatch in mm*)

I = Incompatible

NR = Not recommended (*diametral mismatch in mm*)

|                               |        | Humeral head |         |         |         |         |         |         |         |         |
|-------------------------------|--------|--------------|---------|---------|---------|---------|---------|---------|---------|---------|
|                               |        | ∅ 37 mm      | ∅ 39 mm | ∅ 41 mm | ∅ 43 mm | ∅ 45 mm | ∅ 47 mm | ∅ 49 mm | ∅ 51 mm | ∅ 53 mm |
| Cementless humeral short stem | Size 1 | C            | C       | C       | NR      | NR      | NR      | NR      | NR      | NR      |
|                               | Size 2 | NR           | NR      | NR      | C       | C       | C       | NR      | NR      | NR      |
|                               | Size 3 | NR           | NR      | NR      | NR      | NR      | NR      | C       | C       | C       |

|  |        | Humeral head, eccentric 1.5 mm |         |         |         |         |         |         |         |         |
|--|--------|--------------------------------|---------|---------|---------|---------|---------|---------|---------|---------|
|  |        | ∅ 37 mm                        | ∅ 39 mm | ∅ 41 mm | ∅ 43 mm | ∅ 45 mm | ∅ 47 mm | ∅ 49 mm | ∅ 51 mm | ∅ 53 mm |
| only cemented and cementless humeral stems | Size 1 | C                              | C       | C       | C       | C       | C       | C       | C       | C       |
|  | Size 2 | C                              | C       | C       | C       | C       | C       | C       | C       | C       |
|  | Size 3 | C                              | C       | C       | C       | C       | C       | C       | C       | C       |
|  | Size 4 | C                              | C       | C       | C       | C       | C       | C       | C       | C       |
|  | Size 5 | C                              | C       | C       | C       | C       | C       | C       | C       | C       |
|  | Size 6 | I                              | C       | C       | C       | C       | C       | C       | C       | C       |
|  | Size 7 | I                              | I       | C       | C       | C       | C       | C       | C       | C       |
|  | Size 8 | I                              | I       | I       | C       | C       | C       | C       | C       | C       |
|  | Size 9 | I                              | I       | I       | I       | C       | C       | C       | C       | C       |

|   |        | Humeral head, eccentric 3.5 mm |         |         |         |         |         |         |         |         |
|---|--------|--------------------------------|---------|---------|---------|---------|---------|---------|---------|---------|
|   |        | Ø 37 mm                        | Ø 39 mm | Ø 41 mm | Ø 43 mm | Ø 45 mm | Ø 47 mm | Ø 49 mm | Ø 51 mm | Ø 53 mm |
| onlay cemented and cementless humeral stems | Size 1 | C                              | C       | C       | C       | C       | C       | C       | C       | C       |
|   | Size 2 | C                              | C       | C       | C       | C       | C       | C       | C       | C       |
|   | Size 3 | C                              | C       | C       | C       | C       | C       | C       | C       | C       |
|   | Size 4 | I                              | C       | C       | C       | C       | C       | C       | C       | C       |
|   | Size 5 | I                              | I       | C       | C       | C       | C       | C       | C       | C       |
|   | Size 6 | I                              | I       | I       | C       | C       | C       | C       | C       | C       |
|   | Size 7 | I                              | I       | I       | I       | C       | C       | C       | C       | C       |
|   | Size 8 | I                              | I       | I       | I       | I       | C       | C       | C       | C       |
|   | Size 9 | I                              | I       | I       | I       | I       | I       | C       | C       | C       |

All onlay humeral heads, both cemented and cementless, are compatible with all onlay humeral baseplates.  
 All inlay humeral stems, both cemented and cementless, are compatible with all humeral augments.  
 All inlay humeral stems, both cemented and cementless, are compatible with the dual taper.  
 All inlay humeral stems are compatible with all humeral inserts.

All humeral stems for fracture and revision are compatible with all augments.  
 All humeral stems for fracture and revision are compatible with the dual taper.  
 All humeral stems for fracture and revision are compatible with all distal cortical screws.

All humeral heads are compatible with the dual taper.

|  |         | Pegged glenoid/Cemented keeled glenoid |                              |                               |                               |
|--|---------|--|------------------------------|-------------------------------|-------------------------------|
|  |         | Size 1<br>Reaming Ø 60/80 mm           | Size 2<br>Reaming Ø 60/80 mm | Size 3<br>Reaming Ø 80/120 mm | Size 4<br>Reaming Ø 80/120 mm |
| Humeral head eccentric 1.5 mm and 3.5 mm | Ø 37 mm | C (15.6 mm)                            | C (20 mm)                    | NR (24.4 mm)                  | NR (28.8 mm)                  |
|  | Ø 39 mm | C (13.6 mm)                            | C (18 mm)                    | NR (22.4 mm)                  | NR (26.8 mm)                  |
|  | Ø 41 mm | C (11.6 mm)                            | C (16 mm)                    | C (20.4 mm)                   | NR (24.8 mm)                  |
|  | Ø 43 mm | C (9.6 mm)                             | C (14 mm)                    | C (18.4 mm)                   | NR (22.8 mm)                  |
|  | Ø 45 mm | NR (7.6 mm)                            | C (12 mm)                    | C (16.4 mm)                   | C (20.8 mm)                   |
|  | Ø 47 mm | NR (5.6 mm)                            | C (10 mm)                    | C (14.4 mm)                   | C (18.8 mm)                   |
|  | Ø 49 mm | NR (3.6 mm)                            | C (8 mm)                     | C (12.4 mm)                   | C (16.8 mm)                   |

|  |         |             |           |             |             |
|--|---------|-------------|-----------|-------------|-------------|
|  | Ø 51 mm | NR (1.6 mm) | NR (6 mm) | C (10.4 mm) | C (14.8 mm) |
|  | Ø 53 mm | I           | NR (4 mm) | C (8.4 mm)  | C (12.8 mm) |

|  |         | Augmented pegged glenoid      |                               |                               |                               |
|--|---------|-------------------------------|-------------------------------|-------------------------------|-------------------------------|
|  |         | 8/16°<br>Size 1<br>right/left | 8/16°<br>Size 2<br>right/left | 8/16°<br>Size 3<br>right/left | 8/16°<br>Size 4<br>right/left |
| Humeral head eccentric 1.5 mm and 3.5 mm | Ø 37 mm | C (15.6 mm)                   | C (20 mm)                     | NR (24.4 mm)                  | NR (28.8 mm)                  |
|  | Ø 39 mm | C (13.6 mm)                   | C (18 mm)                     | NR (22.4 mm)                  | NR (26.8 mm)                  |
|  | Ø 41 mm | C (11.6 mm)                   | C (16 mm)                     | C (20.4 mm)                   | NR (24.8 mm)                  |
|  | Ø 43 mm | C (9.6 mm)                    | C (14 mm)                     | C (18.4 mm)                   | NR (22.8 mm)                  |
|  | Ø 45 mm | NR (7.6 mm)                   | C (12 mm)                     | C (16.4 mm)                   | C (20.8 mm)                   |
|  | Ø 47 mm | NR (5.6 mm)                   | C (10 mm)                     | C (14.4 mm)                   | C (18.8 mm)                   |
|  | Ø 49 mm | NR (3.6 mm)                   | C (8 mm)                      | C (12.4 mm)                   | C (16.8 mm)                   |
|  | Ø 51 mm | NR (1.6 mm)                   | NR (6 mm)                     | C (10.4 mm)                   | C (14.8 mm)                   |
|  | Ø 53 mm | I                             | NR (4 mm)                     | C (8.4 mm)                    | C (12.8 mm)                   |

All onlay humeral baseplates and all humeral augments are compatible with all humeral inserts.  
 The designation of humeral inserts defines the diameters of compatible glenospheres.

All glenospheres are compatible with all glenoid baseplates:

- cementless glenoid baseplates,
- cementless glenoid baseplates with central screw hole,
- cementless augmented glenoid baseplates with central screw hole.

All glenoid baseplates are compatible with the peripheral peg and all locked cortical screws and compression cortical screws.

All glenoid baseplates with central screw hole are compatible with all central screws and the obturator.

**7 - Undesirable effects and complications**

- Debonding of one or more prosthetic components
- Displacement of one or more prosthetic components
- Disassembly of one or more prosthetic components
- Rupture of one or more prosthetic components
- Deformation of prosthetic components

- Excessive wear of prosthetic components
- Fracture
- Infection
- Pain
- Luxation or subluxation
- Contracture
- Elongation or shortening of the arm
- Reduction of movement amplitude (impingement)
- Bone remodeling (notching)
- Allergies to materials present in the implant

### **8 - Sterilization**

This implant is supplied in a sterile double blister pack, further to sterilization by ionizing gamma radiation with a minimum rating of 25 kGy. The implant must be:

- Kept in its original packaging,
- Stored in a clean and dry location,
- Protected against light, extreme temperatures and sources of ionizing radiation.

It is essential that the expiry date and the integrity of the packaging be checked before opening, in order to detect any trace of deterioration. Damage to the packaging may indicate that the implant is no longer sterile or has been damaged. In this case, the implant must be returned to MOVE-UP, or destroyed in accordance with regulations in force. The medical device and its packaging are single-use items and must not be re-used under any circumstances. The medical device and its packaging are not suitable for re-sterilization, either by the user or by a healthcare establishment.

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